

Your pneumatic compression device travel guide.

Tips for traveling with your Tactile Medical pump.

Carry-on Bag

If you're bringing your pump in your carry-on, use a small, hard-sided rolling suitcase that can fit under the seat in front of you. Also:

- Include your power cord and any accessories.*
- Bring your Rx Prescription card with you.
- Check with your airline to ensure your medical device will not be counted as a carry-on item. This will enable you to bring other items on board.
- Ensure your carry-on bag is the right size. FAA regulations limit bags to 45 linear inches, which equals the total of the height, width, and length of your bag. Check with your airline for their specific measurements.

Checked Bag

If you're packing your pump in a checked bag, we recommend using a hard-sided suitcase. Consider wrapping your compression garments around your pump to protect it. Keep in mind that if your bag is checked as a medical bag, most airlines will not allow other items inside, such as clothing or toiletries. Also:

- Include your power cord and any accessories.*
- Bring your Rx Prescription card with you.
- Ask your airline carrier to place a "FRAGILE" sticker on your bag to ensure it is handled with care.

Note: The warranty on your pump is voided if your pump becomes damaged during handling by the airline.

Accessories

*Pack applicable accessories based on your prescribed treatment.

FLEXITOUCH® PLUS
BILATERAL PORT
ADAPTER



PORT CAP



PORT PLUG



NOTE: The bilateral port adapter is not needed if you've been prescribed unilateral treatment.



Port Plug



Bilateral Port Adapter +
Port Cap + Port Plug



Bilateral Port Adapter +
Port Cap



Power Cord

Prescribed Treatment

Entre® Plus	✓			✓
Flexitouch Plus Bilateral Lower Extremity			✓	✓
Flexitouch Plus Bilateral Upper Extremity		✓		✓


Packing for Travel Videos



Flexitouch Plus Lower
Extremity Garments:
Packing Your Checked Bag




Flexitouch Plus Vest and
Head and Neck Garment:
Packing Your Checked Bag

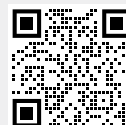


Entre Plus Garments:
Packing Your
Checked Bag



Flexitouch Plus Vest
and Arm Garments:
Packing Your Checked Bag

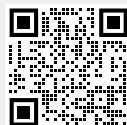


Flexitouch Plus:
Packing your Carry-on

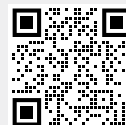


Entre Plus:
Packing your Carry-on

Additional Travel Resources

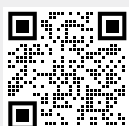

Preparing your
garments for travel.

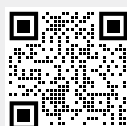

Access your User Guide any time via
Kylee™ your personal lymphedema app,
or online here.

TSA Checkpoint Screening Tips

Use these suggestions to help you get through the security checkpoint as smoothly as possible:

1. Have your TSA Notification Card, otherwise referred to as a Disability Card, ready to show the TSA officer.
2. Inform the TSA officer that you are traveling with your pump and where it is located in your baggage.
3. Remove your pump from your bag for screening; the x-rays will not damage your device.

Need a TSA Notification Card?

This card is used to describe your health condition, disability, or medical device discreetly to a TSA officer.



To request one, scan this QR code with your mobile phone. You can also email your request to TSA-ContactCenter@tsa.dhs.gov, or visit tsa-notification-card.pdfFiller.com.

Note: Devices such as CPAPs, BiPAPs and APAPs are allowed in carry-on bags but must be removed from the carrying case and undergo x-ray screening. For more information regarding portable electronic devices, refer to FAA regulations.

Tactile Medical
3701 Wayzata Blvd, Suite 300
Minneapolis, MN 55416 USA

tactilemedical.com

Customer Care
Text or Call: 612.355.5100
Toll Free Phone: 833.3TACTILE (833.382.2845)
Fax: 612.355.5101 / Toll Free Fax: 866.435.3949
Email: customerservice@tactilemedical.com
Hours: 7 a.m. to 7 p.m. CT, Monday–Friday

Tactile
MEDICAL®
HEALING RIGHT AT HOME