

Traveling with your Tactile Medical pump and garments.

Scan to watch videos on packing your suitcase and learn more about traveling with your Tactile Medical pump.



Flexitouch® Plus Lower Extremity Garments



Leg: Prepare your leg garment(s) as if it was fitted to your leg, unzip and place the hoses inside. Once the hoses are inside, close the zipper. If you've been prescribed bilateral treatment, pack the port cap and port adapter. Include your power cord in your suitcase.

Trunk: Prepare your trunk garment as if you had it on. Coil the hoses onto one side of the trunk garment, and then fold it in half.

Flexitouch Plus Upper Extremity Garments



Vest: Prepare your Vest garment as if you had it on. Coil the hoses onto one side of the vest and fold it in half. Include your power cord in your suitcase.

Arm: Prepare your arm garment(s) as if it was fitted to your arm. Unzip the garment and place the hoses inside, then close the zipper. If you've been prescribed bilateral treatment, pack your port cap, port plugs and port adapter.

*Hose configuration shown is for bilateral full arm and core treatment. If you are not prescribed a trunk garment for your treatment, port 1 on the top row of your bilateral adapter will remain open and will not affect performance.

Head and Neck: Flatten the garment. Coil the hose and place it inside.

Note: It may be helpful to use your garments as padding around your pump. You may also place clothing between your pump and the edge of your suitcase for additional cushioning.

Nimbl™ Garment



ARM

Arm: Flatten the garment and coil the hoses into your garment. Fold it in half or thirds depending on the length of your garment. Include your power cord and optional rechargeable battery in your suitcase.

Note: Would you like to take Nimbl with you? The Nimbl Travel Bag is specially designed to carry your controller, garment, and accessories. The travel bag and rechargeable battery are available for purchase separately at shop.tactilemedical.com.



Scan here to purchase accessories at shop.tactilemedical.com

Entre® Plus Garments



FULL LEG



ARM

Leg and/or Arm: Flatten the garment and coil the hoses into your garment(s). Fold it in half or thirds depending on the length of your garment. If you've been prescribed unilateral treatment, pack the port cap. Include your power cord in your suitcase.

PORT CAP



TSA Checkpoint Screening Tips

Use these suggestions to help you get through the security checkpoint as smoothly as possible:

1. Have your TSA Notification Card, otherwise referred to as a Disability Card, ready to show the TSA officer.
2. Inform the TSA officer that you are traveling with your pump and where it is located in your baggage.
3. Remove your pump from your bag for screening; the x-rays will not damage your device.

Need a TSA Notification Card?

This card is used to describe your health condition, disability, or medical device discreetly to a TSA officer.

To request one, email your request to TSA-ContactCenter@tsa.dhs.gov, or visit tsa-notification-card.pdfFiller.com.

Note: Devices such as CPAPs, BiPAPs and APAPs are allowed in carry-on bags but must be removed from the carrying case and undergo x-ray screening. For more information regarding portable electronic devices, refer to FAA regulations.

Tactile Medical
3701 Wayzata Blvd, Suite 300
Minneapolis, MN 55416 USA

tactilemedical.com

Customer Care
Text or Call: 612.355.5100
Toll Free Phone: 833.3TACTILE (833.382.2845)
Fax: 612.355.5101 / Toll Free Fax: 866.435.3949
Email: customerservice@tactilemedical.com
Hours: 7 a.m. to 7 p.m. CT, Monday-Friday

Tactile
MEDICAL
HEALING RIGHT AT HOME