

Your pneumatic compression device travel guide.

Tips for traveling with your Tactile Medical pump.

Carry-on Bag

If you're bringing your pump in your carry-on, use a small, hard-sided rolling suitcase that can fit under the seat in front of you. Also:

- Include your power cord and any accessories.*
- Bring your Rx Prescription card with you.
- Check with your airline to ensure your medical device will not be counted as a carry-on item. This will enable you to bring other items on board.
- Ensure your carry-on bag is the right size. FAA regulations limit bags to 45 linear inches, which equals the total of the height, width, and length of your bag. Check with your airline for their specific measurements.
- The Nimbl™ Travel Bag is designed to carry your controller, garment, and accessories. You can purchase this bag at shop.tactilemedical.com.

Checked Bag

If you're packing your pump in a checked bag, we recommend using a hard-sided suitcase. Consider wrapping your compression garments around your pump to protect it. Keep in mind that if your bag is checked as a medical bag, most airlines will not allow other items inside, such as clothing or toiletries. Also:

- Include your power cord and any accessories.*
- Bring your Rx Prescription card with you.
- Ask your airline carrier to place a "FRAGILE" sticker on your bag to ensure it is handled with care.

Note: The warranty on your pump is voided if your pump becomes damaged during handling by the airline.

Accessories

*Pack applicable accessories based on your prescribed treatment.

FLEXITOUCH® PLUS
BILATERAL PORT
ADAPTER



PORT CAP



PORT PLUG



NOTE: The bilateral port adapter is not needed if you've been prescribed unilateral treatment.



Prescribed Treatment

Port Plug

Bilateral Port Adapter +
Port Cap + Port Plug

Bilateral Port Adapter +
Port Cap

Power Cord

Entre® Plus	✓			✓
Flexitouch Plus Bilateral Lower Extremity			✓	✓
Flexitouch Plus Bilateral Upper Extremity		✓		✓
Nimbl				✓

Optional Accessories



Purchase separately at
shop.tactilemedical.com

NIMBL RECHARGEABLE
BATTERY



NIMBL
TRAVEL BAG



Scan to watch videos on packing your suitcase and learn more about traveling with your Tactile Medical pump.



TSA Checkpoint Screening Tips

Use these suggestions to help you get through the security checkpoint as smoothly as possible:

1. Have your TSA Notification Card, otherwise referred to as a Disability Card, ready to show the TSA officer.
2. Inform the TSA officer that you are traveling with your pump and where it is located in your baggage.
3. Remove your pump from your bag for screening; the x-rays will not damage your device.

Need a TSA Notification Card?

This card is used to describe your health condition, disability, or medical device discreetly to a TSA officer.

To request one, email your request to TSA-ContactCenter@tsa.dhs.gov, or visit tsa-notification-card.pdfFiller.com.

Note: Devices such as CPAPs, BiPAPs and APAPs are allowed in carry-on bags but must be removed from the carrying case and undergo x-ray screening. For more information regarding portable electronic devices, refer to FAA regulations.

Tactile Medical
3701 Wayzata Blvd, Suite 300
Minneapolis, MN 55416 USA

tactilemedical.com

Customer Care
Text or Call: 612.355.5100
Toll Free Phone: 833.3TACTILE (833.382.2845)
Fax: 612.355.5101 / Toll Free Fax: 866.435.3949
Email: customerservice@tactilemedical.com
Hours: 7 a.m. to 7 p.m. CT, Monday–Friday

Tactile
MEDICAL®
HEALING RIGHT AT HOME