

Care that travels with you: Tips for bringing your Tactile Medical pneumatic compression device on the go.

Your compact device was built for everyday life, even travel. Keep this checklist handy when you pack for your next adventure.

Carry-on bag

- **Check your airline's policy**

Most airlines don't count medical devices toward your carry-on limit, but confirm with your airline ahead of time so you can bring all your bags on board comfortably.

- **Find a sturdy bag**

We highly recommend packing your pump in a Tactile Medical bag or a dedicated, hard-sided bag so it stays safely with you in the cabin.

- **If traveling with a Nimbl rechargeable battery**

For safety reasons, airlines require the battery to stay in your carry-on bag.

Checked bag

- **Protect your device**

If you choose to pack your pump in a checked bag, make sure to wrap it tightly in your compression garments for extra padding and always ask the ticketing agent for a "FRAGILE" sticker. Baggage handling can be rough, and transit damage isn't covered by your warranty.

- **If packing a free medical bag**

If you're checking your device as a "medical bag," airlines usually ask that you pack only your medical equipment inside. You'll need to keep your clothes and toiletries in a separate suitcase.

Optional accessories —
for wherever life takes you.



NIMBL™
TRAVEL BAG



NIMBL
RECHARGEABLE
BATTERY



FLEXITOUCH® PLUS
BAG

shop.tactilemedical.com.

A few friendly reminders.

Pack the essentials. Remember to include your power cord, accessories, and prescription card.

Check out our optional travel accessories. You can find a bag that fits your controller and accessories perfectly.

These accessories are optional and not covered by insurance. They are not necessary to receive effective therapy.



Pack with confidence. Scan here to see the latest videos on how to travel with your pump.



Purchase separately at shop.tactilemedical.com.

Accessories

Pack applicable accessories based on your prescribed treatment.



FLEXITOUCH PLUS BAG WITH GARMENTS AND APPLICABLE ACCESSORIES



POWER CORD



PORT PLUG



PORT CAP



BILATERAL
PORT ADAPTER

Nimbl/Entre® Plus	✓			
Flexitouch Plus Bilateral Lower Extremity	✓		✓	✓
Flexitouch Plus Bilateral Upper Extremity	✓	✓	✓	✓

Transportation Security Administration (TSA) checkpoint screening tips

Navigating security with medical equipment is easier than you might think. Just follow these three simple steps:

1. Bring your free TSA notification card (also known as a disability card) to show the agent.
2. Give the agent a friendly heads-up that you have a medical pump packed in your bag.
3. Place the pump in a bin for screening. The X-rays will not harm or damage your device in any way.

Need a TSA Notification Card?

This free card is the easiest way to discreetly communicate your health condition or medical device to a TSA officer.



Scan here to download the card or visit [tsa.gov/travel/tsa-cares](https://www.tsa.gov/travel/tsa-cares).

Tactile Medical

3701 Wayzata Blvd, Suite 300
Minneapolis, MN 55416 USA

[tactilemedical.com](https://www.tactilemedical.com)

Customer Care

Text or Call: 612.355.5100
Toll Free Phone: 833.3TACTILE (833.382.2845)
Fax: 612.355.5101 / Toll Free Fax: 866.435.3949
Email: customerservice@tactilemedical.com
Hours: 7 a.m. to 5:30 p.m. CT, Monday–Friday

Tactile
MEDICAL®